

OsiriX Instructions

Import Images



Choose Import in menu bar or drag files to program icon in dock- copy links not files
Note: if you move the files you will need to reimport them as the program links to the file.



Can add names in comment section. Can move comment section

Customize Tool Bar-


Open an image so you are in the correct toolbar.
Choose Customize toolbar
Add flip horizontal

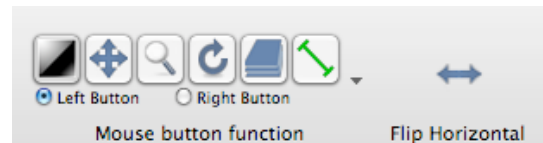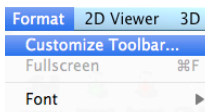

Viewing Images

Double click to open one
Hold Command key down and choose multiple images, double click
CT open 4 images
MR open 8 images

CT- Flip Images if needed. All rights on the left hand side of screen as if you are looking at a patient.

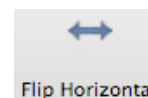


MR- Auto Synchronize series so all images move at same time.

Un sync CT, Sync MR


Propagate to change all image setting at the same time (zoom, contrast, etc).



Customize Image

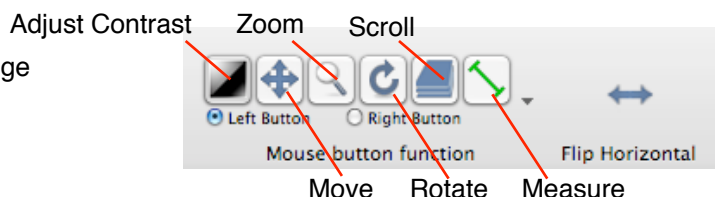


Adjust Contrast- Change WL/WW to CT Bone or Default or use above tool
Reverse White and Black- Change CLUT to B/W Inverse

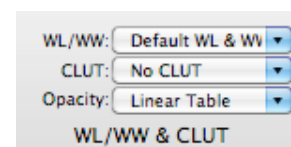

Scroll- 4 ways

Top Slider
Mouse- choose from toolbar- can be Right or Left, your preference. Right most common
Left Arrow key
Scroll wheel on mouse.

Explode view- Click on green +. To go back click on green -

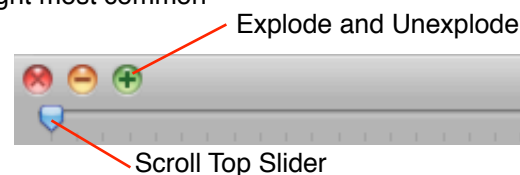


Command L sets default contrast

Control Key tells you what scan you have

Command R resets image back to start

Can do advanced image viewing
2D MPR- MultiPlaner Reconstructions
3D Volume Rendering

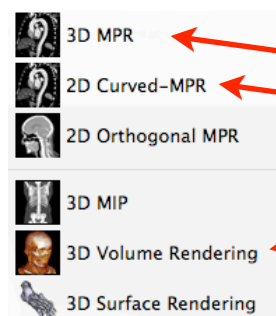

Dice and Slice

Make PanX

3D View

If iCAT and does not know slice interval for MPR- copy Pixel Y resolution, paste, and make negative.